

Fleet Advisory 2025-05

PSC deficiency

Port: Taixing, China

Date: 13 May 2025

Code 18401	The oxygen cylinder of the oxygen resuscitator in the ship's infirmary leakage. Comments: - Minor air leak was noticed between the air regulator and humidifier bottle when cylinder valve (40 litre) for checking the air pressure was opened. Probably the nut connecting the humidifier bottle and air regulator was not properly tightened after last filling up of the water in the bottle during last routine inspection.  - Ensure fittings/connections are properly tightened after routine inspection/maintenance. - Check for any air leakage after completion of the maintenance by opening the air cylinder valve.
-----------------------	---

Above has been read and understood.

C/O

2/O

3/O

D/C

Verified by: Master

Please file signed copy in the Share Point 3.2.3 Training Folder